Reprocessing of Reusable Non-Sterile Instruments

Instructions For Use

Part Number: 400-494 | Rev. 5 | 10 June 2021

CONTENTS

ENGLISH (en)	3
FRANCAIS (fr)	7
DEUTSCH (de)	13
ESPAÑOL (es)	17
ITALIANO (it)	22
PORTUGUÊS (pt)	27
NEDERLANDS (nl)	32
DANSK (da)	37
SVENSKA (sv)	41
SUOMI (fi)	46
NORSK (no)	50
ΕΛΛΗΝΙΚΑ (el)	54
ČESKÉ (cs)	60
MAGYAR (hu)	64
POLSKI (pl)	70
SLOVENŠČINA (sl)	75
TÜRKÇE (tr)	79
한국어 (ko)	85
SYMBOLS	90

General Information

Please read carefully before using this product.

Instruments

These instructions apply to reusable non-sterile surgical instruments supplied by MatOrtho® and intended for reprocessing in a health care facility setting.

These instructions are intended for use only by persons with the required specialist knowledge and training.

Warning

Instruments used to implant orthopaedic prostheses do not have an indefinite life. All reusable instruments are subjected to repeated stresses related to bone contact, impaction and routine cleaning and sterilisation processes. It is essential that the surgeon and operating theatre staff are fully conversant with the appropriate surgical technique for the instruments and associated implants, if any.

CAUTION: These instructions do not apply to single use devices.

Warnings.	<ul style="list-style-type: none">• Follow instructions and warnings as issued by manufacturers of any decontaminants, disinfectants and cleaning agents used. Wherever possible avoid the use of mineral acids and harsh, abrasive agents.• No part of the process shall exceed 140°C.• Some sensitive materials (e.g. Aluminium) are damaged by high alkaline solutions (pH >10).• Devices with long, narrow cannula, hinges and blind holes require particular attention during cleaning. <p>Note: when reprocessing medical devices, always handle with care, wearing protective clothing, gloves and eyewear in accordance with local Health and Safety procedures.</p>
Limitations on reprocessing.	<ul style="list-style-type: none">• Repeated processing has minimal effect on these instruments.• End of life is normally determined by wear and damage in use.• Any specific limitations on the number of reprocessing cycles shall be made available with the instrument.

Instructions.

From point of use.	<ul style="list-style-type: none"> Wherever possible, do not allow blood, debris or bodily fluids to dry on instruments. For best results and to prolong the life of the instrument, reprocess immediately after use. If it is not possible to reprocess immediately, use an enzymatic foam spray cleaner to help prevent soil from drying.
Preparation for decontamination.	<ul style="list-style-type: none"> Reprocess all instruments as soon as it is reasonably practical following use. Only disassemble where intended by MatOrtho®. Where instructions for disassembly are required, these are available with the operative technique and/or as separate instructions provided with the instruments.
Cleaning: Automated.	<ul style="list-style-type: none"> Use only CE marked or validated washer-disinfector machines and low-foaming, non-ionising cleaning agents and detergents; following the manufacturers' instructions for use, warnings, concentrations and recommended cycles. Load instruments carefully, with any box joints and hinges open and so that any holes or recesses in the instruments can drain. Place heavy instruments with care into the bottom of containers, taking care not to overload wash baskets. Place instruments with concave surfaces facing down, to prevent pooling of water. Where available, use appropriate attachments to flush inside reamers and devices with lumens or cannula. Ensure that soft, high purity water which is controlled for bacterial endotoxins is used in the final rinse stage. <p>Note: Automated cleaning may not be suitable for all lumens and cannula, in which case clean manually with a water jet gun if available, and an appropriate brush and stiletto,) that reaches into the depth of the feature. After manual cleaning, pass all instruments through an automated cleaning cycle, to achieve disinfection.</p> <p>Note: These instructions have been validated using a washer-disinfector cycle to include two cold rinses at <35°C, a detergent cycle and a rinse cycle, both at >50°C, a disinfection cycle operating at a temperature of between 80°C and 87°C for a minimum holding time of 1 minute (actual holding time in excess of 2 minutes 50 seconds) and a 20 minute drying cycle. The detergent used was a low foaming, non-ionising spray cleaner (max 12pH), and the rinse aid was a neutral pH low foaming, non-ionic surfactant with isopropyl alcohol.</p>

<p>Cleaning: Manual.</p>	<p>Manual cleaning is not advised if an automatic washer-disinfector is available. If this equipment is not available, use the following process:</p> <ol style="list-style-type: none"> 1. Use a double sink system (wash/rinse) dedicated for instrument cleaning and not used for hand washing). 2. Ensure that the water temperature does not exceed 35°C. 3. In the first sink, keeping the instrument submerged with an autoclavable brush, apply CE marked cleaning solution to all surfaces until all soil has been removed. Pay particular attention to serrations, teeth, ratchets and hinges. Always brush away from the body and avoiding splashing. Ensure rongeurs and hinged instruments are thoroughly cleaned in both open and closed positions. 4. In the second sink, rinse instruments thoroughly with soft, high purity water which is controlled for bacterial endotoxins, so that the water reaches all parts of the instrument. Carefully hand dry or use a drying cabinet. <p>Note: Manual cleaning is NOT a disinfection process: When manual cleaning is used it may not be possible to disinfect the device prior to further handling.</p>
<p>Cleaning: Inspection.</p>	<ul style="list-style-type: none"> • After cleaning, visually inspect all surfaces, cannulations, ratchets, joints, holes and lumens for the complete removal of soil and fluids. If any soil or fluid is still visible, return the instrument for repeat decontamination.
<p>Maintenance.</p>	<ul style="list-style-type: none"> • Apply surgical grade lubricants to hinges, joints and moving parts, as per the lubricant manufacturer's instructions.
<p>Inspection and function testing.</p>	<ul style="list-style-type: none"> • Visually inspect and check: All instruments for damage and wear. Cutting edges are free of nicks and present a continuous edge. Jaws and teeth align correctly. All articulated instruments have a smooth movement without excess play. Locking mechanisms (such as ratchets) fasten securely and close easily. Long, slender instruments are not distorted. Any component parts fit and assemble correctly with the mating components. • Remove for repair or replace any blunt, worn out, flaking, fractured or damaged instruments. <p>Note: If an instrument is returned to the manufacturer/supplier, the instrument must be decontaminated and sterilised and be accompanied with the relevant documented evidence.</p>
<p>Packaging.</p>	<ul style="list-style-type: none"> • All instruments to be packed following local protocols.

Sterilisation.	<i>Method:</i>	Moist heat sterilisation according to ANSI/AAMI/ ST 79.
	Cycle:	Pre-Vacuum (Pre-Vac).
	Temperature:	132°C (270°F)
	Exposure Time:	4 minutes (Minimum)
	Drying Time:	30 minutes (Minimum in chamber).
	<i>Method:</i>	Moist heat sterilisation according to ISO 17665
	Cycle:	Saturated steam with fractional forced air removal.
	Temperature:	134-137°C (270-277°F).
	Exposure Time:	3 minutes (Minimum)
	Drying Time:	30 minutes (Minimum in chamber)
	<ul style="list-style-type: none"> • Use only a CE marked or validated vacuum autoclave. Always follow the instructions of the machine manufacturer. • When sterilising multiple instruments in one autoclave cycle, ensure that the steriliser manufacturer's stated maximum load is not exceeded. • Ensure instruments are dry before sterilisation. 	
Storage.	<ul style="list-style-type: none"> • Ensure instruments are dry before storage, and store in dry, clean conditions at ambient room temperature. 	
Additional Information.	<ul style="list-style-type: none"> • Other forms of cleaning (i.e. Ultrasonic) and Sterilisation (i.e. Low Temperature Steam and Formaldehyde, Ethylene Oxide and Gas Plasma) are available. Always follow the instructions for use as issued by the manufacturer and consult them if in any doubt over the suitability of any process used. 	
Manufacturer.	<ul style="list-style-type: none"> • See brochure for telephone and address of local representative or telephone +44 1372 224 200. 	

Note:

It is the responsibility of the reprocessor to ensure that the reprocessing is performed using equipment, materials and personnel in the reprocessing facility to achieve the desired results. This requires validation and routine monitoring of the processes. Likewise any deviation by the reprocessor from the instructions provided must be properly evaluated for effectiveness and potential adverse consequences.

The latest version of these instructions for use can be found on the MatOrtho® website at <https://info.matortho.com/reg/400-494.pdf>.

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	<p>MatOrtho Limited 19/20 Mole Business Park Randalls Road Leatherhead Surrey KT22 7BA England</p>	<p>T: +44 1372 224200 E: info@matortho.com W: www.matortho.com</p>
	<p>MatOrtho Ireland Limited The Black Church Saint Mary's Place D07 P4ax Dublin Republic of Ireland</p>	<p>E: AR@matortho.eu</p>
		
		

— |

| —

— |

| —

Forever Active

MatOrtho[®] Limited | 19/20 Mole Business Park | Randalls Road
Leatherhead | Surrey | KT22 7BA | United Kingdom

T: +44 (0)1372 224200 | info@MatOrtho.com

For more information visit www.MatOrtho.com

CE 1434

Part Number 400-494 | Rev.5 | 10 June 2021